


شیوادنت Program Description 	Time (min) زمان بر حسب (دقیقه)			Time (min) زمان بر حسب (دقیقه)			vacum (وکیوم)	Firing Program °C برنامه پخت بر حسب سانتی گراد					
	Dry	Pre Heat	Vac Hold	Hi Temp Hold	Hold	Cool	Set Point (in Hg)	Set Temp		Temperature			
								Idle	Hi Temp	Vac Start	Vac Stop	Heat Rate C/min°	Night
Paste Opaque اپک خمیری	5	3	0	0	0	0	29	500	975	500	975	100	100
Powder Opaque اپک پودری	3	3	0	0	0	0	29	650	970	650	970	70	100
Margin - Improved	5	5	0	1	0	0	29	650	965	650	965	70	100
Final Margin (with overglaze)	3	3	0	0.5	0	0	0	650	935			70	100
Final Margin (with natural glaze)	3	3	0	0.5	0	0	0	650	945			70	100
Op Dentin/Denint/Mod پودر دنتین	5	5	0	0	0	0	29	650	960	650	960	55	100
Enamels (انسیزال)	5	5	0	0	0	0	29	650	960	650	960	55	100
Natural Glaze	3	3	0	0.5	0	0	0	650	945			70	100
Overglaze	3	3	0	0.5	0	0	0	650	935			70	100
Add-On	5	5	0	0	0	0	29	650	940	650	940	70	100